




How to Reprocess Eye Protection

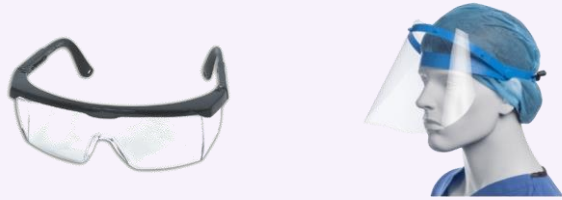

General Notes

- Remove and reprocess if visibly soiled
- Perform reprocessing outside patient room
- Wash hands if eye protection is touched
- Discard eye protection if damaged

1. Perform hand hygiene
2. Put on gloves 
3. Use an **alcohol wipe** across the entire **INNER** surface
4. Use a hospital disinfectant (**SaniWipe AF3**) to wipe the entire **OUTER** surface
5. Allow to air dry
6. Place in brown paper bag till next use 
7. Remove gloves and perform hand hygiene