

Airborne Precautions with ILC Dover PAPR

Donning Sequence

Pre-Check

- ☐ Inspect blower unit for damage.
- ☐ Ensure both HEPA filters are securely attached to each side of the blower.
- ☐ Turn blower ON: A short beep should sound and the red LED light should illuminate continuously. Audible beeping and blinking red LED light indicates low battery. Turn OFF.
- ☐ Check air flow: attach air flow meter to blower hose connector, hold upright, turn blower ON. Ensure upper portion of the black cone is above the indication line on flow meter, indicating ready for use. Turn OFF. Return flow meter to storage bag.

Donning PPE

- ☐ Perform Hand Hygiene. Don procedure facemask to wear under hood.
- ☐ Check that breathing hose is securely attached to male connectors on head cover and PAPR blower. If not attached, tightly screw breathing hose to the male connections.
- ☐ Snap buckle closed around waist and properly adjust waist strap to snug fit with PAPR blower at low back.
- ☐ Don isolation gown over PAPR belt.
- ☐ Don double gloves.
- ☐ Turn PAPR power ON.
- ☐ Open clear hood opening and place hood over head.
- ☐ Tuck inner skirts (2) into PPE gown.
- ☐ Smooth outer skirt in place.
- ☐ Have Buddy check PAPR hood and skirt.

Doffing PPE with ILC Dover PAPR

Airborne Precautions with ILC Dover PAPR Doffing Sequence

ALWAYS have a Buddy help you DOFF your PPE & PAPR

- ☐ **REMOVE** outer gloves and gown and throw away inside room.
- ☐ Perform **HAND HYGIENE with alcohol foam** on inner gloves.
- ☐ Step out of the room.
- ☐ **Buddy** in PPE should **WIPE** hood with disinfectant wipes.
- ☐ Unclip PAPR blower unit and hand to Buddy. **Do Not turn off PAPR.**
- ☐ **WIPE** PAPR blower with disinfectant wipes.
- ☐ **REMOVE 2nd set of GLOVES.**
- ☐ Perform **HAND HYGIENE with alcohol foam cleanser.**
- ☐ Put on new gloves.
- ☐ Lean forward and **REMOVE PAPR hood** by lifting up from within hood. Place on cart/bucket.
- ☐ Turn off PAPR blower.
- ☐ **WIPE inside and outside of hood** and PAPR blower with disinfectant.
- ☐ **RETURN** PAPR to storage area and complete log.
- ☐ Perform **HAND HYGIENE** with SOAP and WATER for 20 seconds

